





The first step after purchase (Pairing with each device)


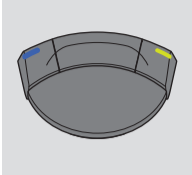
1
At least 2 sec.

Press and hold the SETTING Key. "BLE SET" is displayed.

2

Press the ENTER Key.


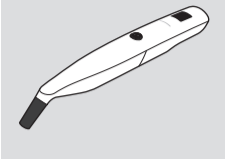
3

Press the TORQUE Key to select the device.
NOTICE : The letters on the screen except "BLE SET" and "PAIRING" flash on and off.






Foot control




Osseo 100+

VarioSurg3




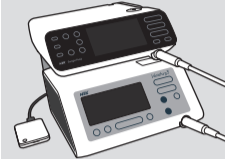

External terminal



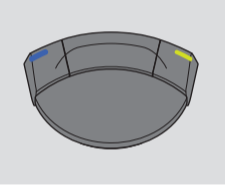


Press the ENTER Key.

▼ Foot control




This operation is not necessary when using Surgic Pro2 for the first time after purchase. If the foot control is unpaired or when using another foot control newly purchased, conduct the following operation.
NOTICE : Paring can be done within 10 minutes after inserting the battery.




Press the ENTER Key.

Approx. 10 sec.

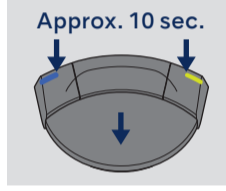


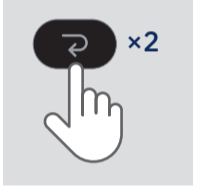
Press and hold the Left and Right buttons and the pedal at the same time.



The control unit makes a beep and then "OK" will appear. Pairing is done. Make sure that the device is connected.


Connected






Press the CANCEL Key to return to the normal screen.


▼ Osseo 100+

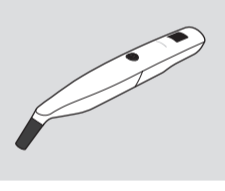




Press the operating key to turn on the power.


At least 3 sec.



Press and hold the operating key.


"PA" is displayed.






The control unit makes a beep and then "OK" will appear. Pairing is done. Make sure that the device is connected.


Connected






Press the CANCEL Key to return to the normal screen.


▼ VarioSurg3

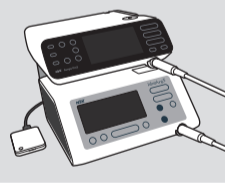


Turn on the power.




Connect the link cable beforehand.

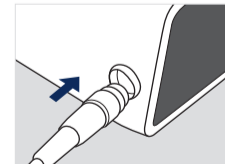


Press and hold the button.

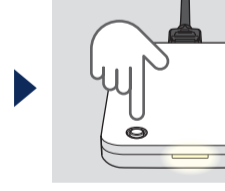


When the LED turns blue, release the button.

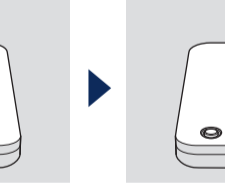




The control unit makes a beep and then "OK" will appear. Pairing is done. Make sure that the device is connected. Also the LED of link module turns green.


Connected



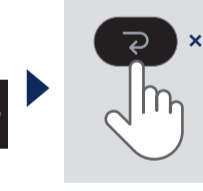


Press the CANCEL Key to return to the normal screen.

▼ External terminal




Launch the App.




Install the App beforehand.


Touch the icon.

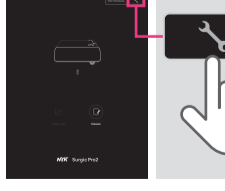


Touch "Start".

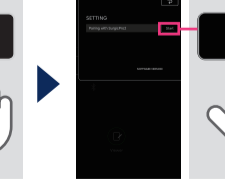


Enter the code "123456" and touch "Pair".

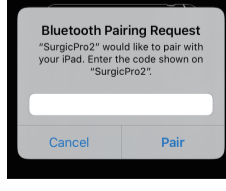


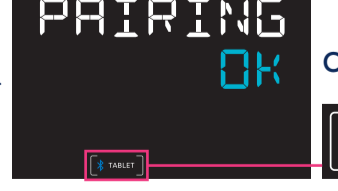

The control unit makes a beep and then "OK" will appear. Pairing is done. Make sure that the device is connected.

Connected



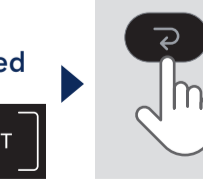

Press the CANCEL Key to return to the normal screen.



www.nsk-dental.com/qr/app-surgicpro2